

This document shows how to connect and use an Ovi-Scan 6 pregnancy scanner (Ovi-Scan 6) with an XR5000 or ID5000 weigh scale indicator.

Note: An XR5000 indicator will allow you to create a custom field for data entry from the Ovi-Scan 6. If you have an ID5000, you should use the preset 'Trait' field for data entry. See below for details.

1 Creating a field for data entry

To set up an XR5000 for use with the Ovi-Scan 6:

1. On the XR5000, create a new field for data entry called 'Preg scan'. To do this, go to **Settings, Information to display and record, New**, then complete the details as below:

2. Optional - You can also set up a new field for data entry to record Foetal Age with options 'Early', 'Mid', 'Late'. To do this, go to **Settings, Information to display and record, New**, then complete the details as below:

To set up an ID5000 for use with the Ovi-Scan 6:

You will use one of the preset fields 'Trait' for data entry. There is no setup required.

2

Connecting the Ovi-Scan 6 to the 5000 indicator (applies to both XR5000 and ID5000 indicator)

1. Connect the Ovi-Scan 6 to the indicator using one of the following serial adaptor cables:

826638 5000 series Cable C16 to 2-port DB9 serial and power C16




818324 5000 series or EziWeigh6/7 to D9 Serial




2. On the indicator's Home screen, select **Settings, Next Page, Connections, Serial Cable**.
3. If using 826638 you will need to select the port which you have plugged the cable in to. If using 818324, the port will always be Port 1.
4. Under whichever port the scanner is connected to, apply the following settings:

XR5000:

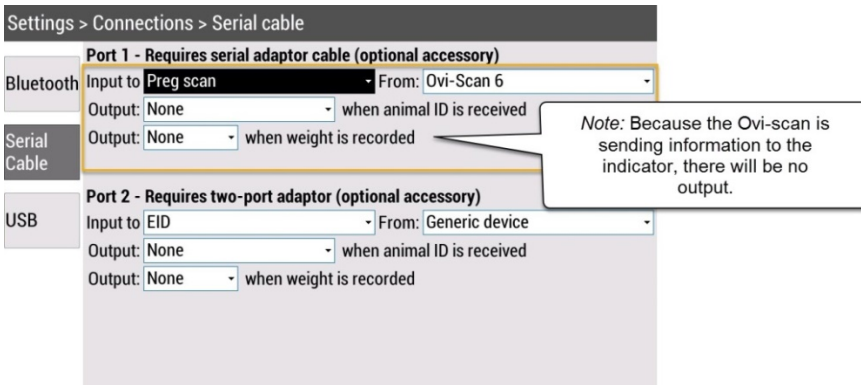

ID5000:

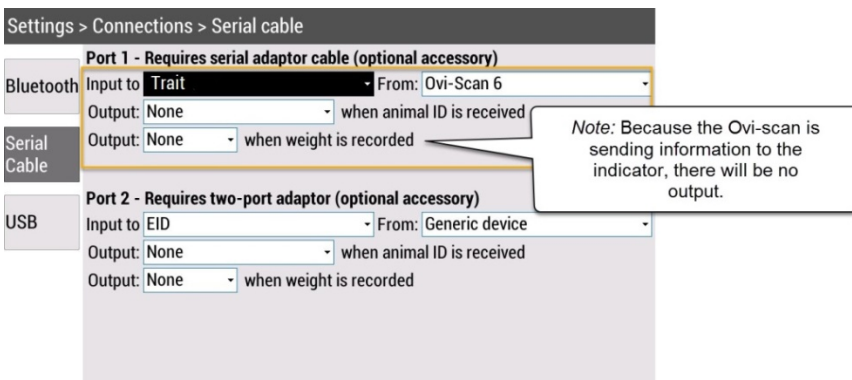

3



Setting up the Weigh screen for use with the Ovi-Scan 6

You can set up the Weigh screen with data entry fields so that you can view and enter data during the session, as required.

To set up the Weigh screen, go to **Settings, Information to display and record**.

Tick the fields that you would like to display on the Weigh screen.

Example:

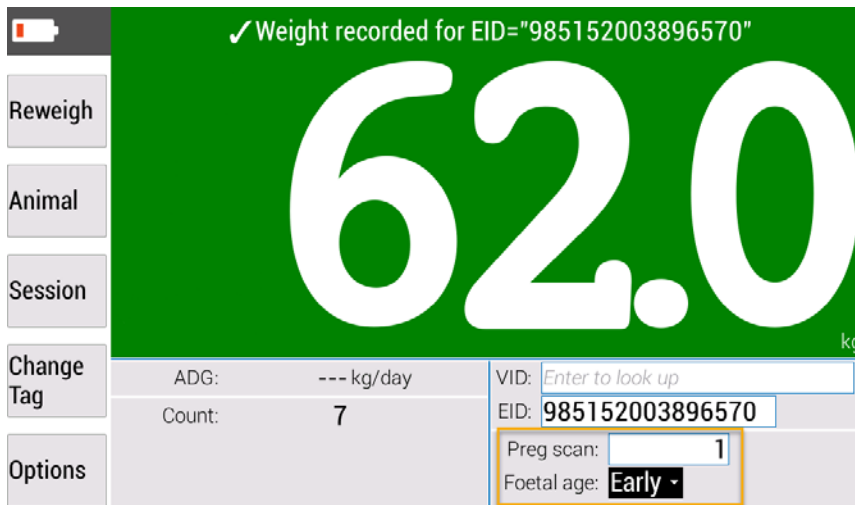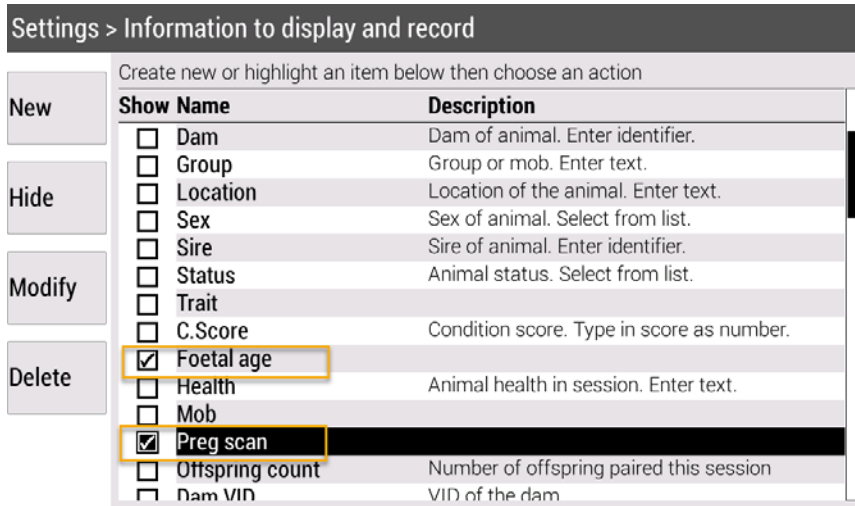

For further information regarding your stock management system, contact Tru-Test Ltd on 1800 621 324 (AU) or 0800 878 837 (NZ)
 For technical support, contact 1800 248 774 (AU) or 0800 AGDATA (0800 243 282) (NZ)